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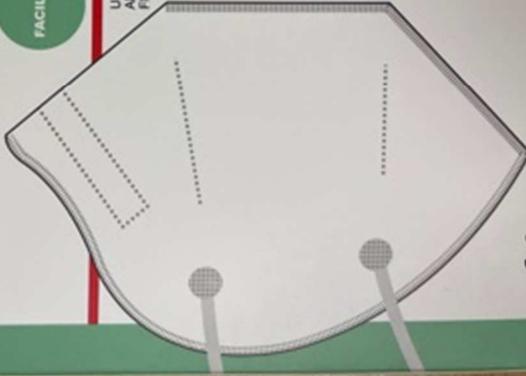
COMFORTABLE
PROTECTIVE
MASK

- FACILITY
- FLIGHTY
- DEFEND
- BREATHER

USE ECO-FRIENDLY NON-WOVEN FABRICS, MELTBLOWN AND OTHER MULTILAYER FABRICS TO EFFECTIVELY RESIST FINE PARTICLES AND DROPLETS IN AIR POLLUTION.

FOUR LAYERS

20 PCS

CE EN149:2001+A1:2009 FFP2 NR D

FUNCTION

WEARING TO COVER AND PROTECT THE MOUTH AND NOSE, CAN EFFECTIVELY PREVENT THE INHALATION OF DUST AND DROPLETS.

USAGE

1. FACE THE MASK WITHOUT THE NOSE CLIP, AND THE NOSE CLIP SHOULD BE ON TOP;
2. HOLD THE MASK ON YOUR FACE AND AGAINST YOUR CHIN;
3. PULL THE STRAPS ON BOTH SIDES AND HANG ON THE EARS, AND ADJUST TILL YOU FEEL COMFORTABLE;
4. PUT YOUR FINGERS IN THE MIDDLE OF THE METAL NOSE CLIP, AND MOVE YOUR FINGERTIPS TO BOTH SIDES UNTIL THE NOSE CLIP IS FULLY PRESSED INTO THE SHAPE OF THE NOSE SLOPE;
5. PUT YOUR HANDS ON YOUR FACE TO CHECK THE TIGHTNESS WITH YOUR FACE.

PRODUCT INFORMATION

PRODUCT NAME: PROTECTIVE MASK

PRODUCT FEATURE AND COMPOSITION: THIS PRODUCT IS LAMINATED FROM MULTIPLE LAYERS OF THIN FABRIC

SIZE: ADULT

MANUFACTURER: ZHONGSHAN CITY ZHONGXIN GIFT CO.,LTD

PRODUCTION ADDRESS: NO.188 ROAD 5 DONGHAI DONGFENG TOWN

ZHONGSHAN GUANGDONG CHINA

IMPLEMENTATION STANDARD: GB 2626-2006

DATE OF MANUFACTURING: SEE INNER BOX CERTIFICATE

VALIDITY: 3 YEARS

PACKING SPECIFICATION: 20 PCS/CASE

STORAGE: STORED IN A VENTILATED, DRY, HYGIENIC AND NON-CORROSIVE GAS PLACE.

CAUTION: PATIENTS WITH DIFFICULTY BREATHING OR CONGENITAL HEART DISEASE,

PLEASE USE WITH CAUTION;

THIS PRODUCT CANNOT BE USED AS A GAS MASK IN A TOXIC ENVIRONMENT.

MADE IN CHINA